

Online store WWW.LYNXGEAR.LV Company: SIA LATLYNX


info@lynxgear.lv, +371 26159668 Reg. No. 45403038162, VAT Reg. No. LV45403038162,

Legal address: Baznīcas iela 8, Aizkraukle, Aizkraukles novads, LV-5101, LATVIJA

Bank Account: LV87UNLA0050021387494 AS SEB Bank, SWIFT: UNLALV2X

ORDER CONFIRMATION

Not Rated Yet

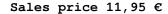





Description


3iX Tick Remover Loop

The 3iX Tick Remover Loop (sometimes called the "TickLasso") is an for safely and effectively removing ticks without innovative tool increasing the risk of disease transmission. Its noose system captures the tick without crushing it, ensuring the entire parasite (including the head) is removed-quickly, cleanly, and with minimal stress to both the tick carrier and the user.

Key Features

- Safe & Complete Removal: Avoids squeezing or stressing the tick, which helps prevent the tick from regurgitating pathogens into the wound.
- Noose System: Easily captures the tick-small or large-while ensuring the head detaches in one piece.
- Prevents Pathogen Transmission: Reduces the risk of infections like Lyme disease or tick-borne encephalitis by gently removing the tick without crushing it.
- Universal Use: Suitable for humans and animals (dogs, cats, livestock), making it a must-have for outdoor enthusiasts, travelers,
- Lightweight & Compact: Weighs only 6 g and measures about 11.5 cm, easily carried in a pocket or first aid kit.

Page 1/3

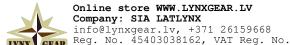

LV45403038162,

Legal address: Baznīcas iela 8, Aizkraukle, Aizkraukles novads, LV-5101, LATVIJA

Bank Account: LV87UNLA0050021387494 AS SEB Bank, SWIFT: UNLALV2X

ORDER CONFIRMATION

ISO 9002 Certified: Developed and manufactured in Sweden according to strict quality standards.


Specifications

• Length: 11.5 cm • Weight: 6 g

How to Use

- 1. Place the noose around the tick's mouthparts-avoid pressing or squeezing the tick.
- Tighten the loop gently, ensuring the tick's body is captured but not crushed.
- 3. Rotate or twist slightly if needed, then pull outward smoothly. This method uses the tick's own barbs to your advantage.
- 4. Dispose of the tick responsibly (e.g., flushing it away, sealing it in tape, or checking it for disease).
- 5. Clean and disinfect the bite area and the tick remover.

Tip: The risk of disease transmission rises the longer a tick remains attached. Always remove the tick as quickly as possible-preferably within 24 hours of attachment-and avoid "home remedies" that might stress the tick.

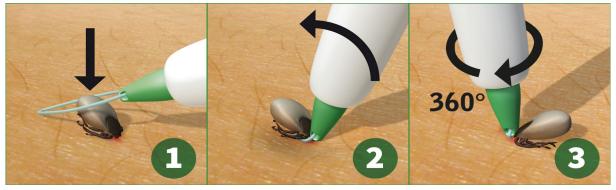

Additional Information

• Further Reading on Tick-Borne Diseases & Prevention

With the 3iX Tick Remover Loop, ticks of all sizes can be removed gently and effectively, reducing discomfort for children, pets, and sensitive areas of the body. Developed and manufactured in Sweden, this tool is a staple for doctors, veterinarians, and everyday outdoor enthusiasts looking for quick and safe tick removal.

Manufacturer:

Karo Pharma AB

Page 2/3



Online store WWW.LYNXGEAR.LV
Company: SIA LATLYNX
info@lynxgear.lv, +371 26159668
Reg. No. 45403038162, VAT Reg. No.
LV45403038162,
Legal address: Baznīcas iela 8,
Aizkraukle, Aizkraukles novads,
LV-5101, LATVIJA
Bank Account: LV87UNLA0050021387494
AS SEB Bank, SWIFT: UNLALV2X

ORDER CONFTRMATION Box 16184, 103 24 Stockholm, Sweden

Page 3/3