




Online store WWW.LYNXGEAR.LV
Company: SIA LATLYNX
info@lynxgear.lv, +371 26159668
Reg. No. 45403038162, VAT Reg. No.
LV45403038162,
Legal address: Baznicas iela 8,
Aizkraukle, Aizkraukles novads,
LV-5101, LATVIJA
Bank Account: LV87UNLA0050021387494
AS SEB Bank, SWIFT: UNLALV2X

ORDER CONFIRMATION

Not Rated Yet

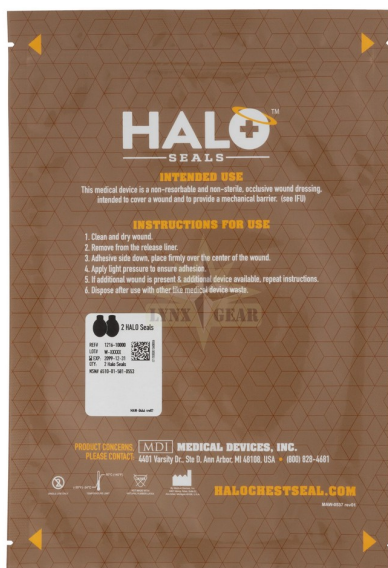

Sales price 29,95 €

Salesprice with discount

Incl. VAT 21%: 5,20 €



Description

First Aid for Open Chest Wounds - Halo Chest Seal

This is non-resorbable and non-sterile, occlusive wound dressing, intended to cover a wound and to provide mechanical barrier.

Features:

- **Excellently adhering hydrophilic gel coating:** Ensures a secure seal, even in the presence of blood or sweat.
- **No valve design:** Eliminates the risk of valve displacement, providing reliable performance.
- **Large tab for easy removal:** Allows for quick and easy removal, even in emergency situations.
- **Versatile use:** Can also be used to secure other wound dressings, adding to its versatility.
- **Clean removal:** No adhesive residue left behind after removal.
- **Notches on all 4 corners:** Designed for easy opening, ensuring fast access when every second counts.



Online store WWW.LYNXGEAR.LV
Company: SIA LATLYNX
info@lynxgear.lv, +371 26159668
Reg. No. 45403038162, VAT Reg. No.
LV45403038162,
Legal address: Baznīcas iela 8,
Aizkraukle, Aizkraukles novads,
LV-5101, LATVIJA
Bank Account: LV87UNLA0050021387494
AS SEB Bank, SWIFT: UNLALV2X

ORDER CONFIRMATION

- **Resealable packaging:** Packaging can be resealed if the product is not immediately used, preserving its sterility.
- **Wide temperature range:** Effective for use in temperatures from -40 °C to +65 °C.
- **Long shelf life:** Can be used for up to 3 years from the date of manufacture.
- **Latex-free:** Safe for use on patients with latex allergies.

The item comes in a pack of 2 pieces. The price also refers to the double pack.

NSN: 6510-01-581-0553

Instructions for use:

- Clean and dry wound
- Remove seal from the release liner
- Adhesive side down, place firmly over the center of the wound
- Apply light pressure
- Dispose after use with other like medical devices.

Brand country - U.S.