



Online store WWW.LYNXGEAR.LV
Company: SIA LATLYNX
info@lynxgear.lv, +371 26159668
Reg. No. 45403038162, VAT Reg. No.
LV45403038162,
Legal address: Baznicas iela 8,
Aizkraukle, Aizkraukles novads,
LV-5101, LATVIJA
Bank Account: LV87UNLA0050021387494
AS SEB Bank, SWIFT: UNLALV2X

ORDER CONFIRMATION

Not Rated Yet



Sales price 0,80 €

Salesprice with discount

Discount -0,40 €

Incl. VAT 21%: 0,14 €

1-4 Weeks



Description

Flexible PU plastic film is used to ensure a super-thin, flexible, washproof plaster. The membrane is moisture vapour permeable, allowing the skin to breathe, without the need for physical perforations, keeping the adhesive securely in place, whilst acting as a complete barrier to water, virus, bacteria and fungi. High-tech acrylic adhesive provides a really secure fixing, but is skin friendly and low allergy.

Kit contents:

- 8 pcs. 4cm x 4cm
- 8 pcs. 7cm x 2cm
- 4 pcs. 7.5cm x 2.5cm

Features:

- Clear.
- Moisture vapour permeable membrane.
- Perfect barrier to water, virus, bacteria and fungi.



Online store WWW.LYNXGEAR.LV
Company: SIA LATLYNX
info@lynxgear.lv, +371 26159668
Reg. No. 45403038162, VAT Reg. No.
LV45403038162,
Legal address: Baznicas iela 8,
Aizkraukle, Aizkraukles novads,
LV-5101, LATVIJA
Bank Account: LV87UNLA0050021387494
AS SEB Bank, SWIFT: UNLALV2X

ORDER CONFIRMATION

- Acrylic, low allergy adhesive.

Instructions for use:

Clean the scar and the area around it before use. Dry the skin completely and apply the patch so that the bandage pad completely covers the injury. Press the patch gently and evenly. Glue the patch without stretching. Use each patch only once.

Country - UK.